

TissueSeal

A division of H.B. Fuller Company

MEDICAL

HISTOACRYL[®] Topical Skin Adhesive

Now on FSS/GSA contract through our partner
MedLog Solutions, LLC.

WHY HISTOACRYL[®] FLEXIBLE?

- Histoacryl Flexible has been designed to combine flexibility, strength, and fast polymerization times, to close lacerations and surgical incisions.
- Tensile strength is significantly higher for Histoacryl Flexible, than the leading competitor.
- Close up to 25 linear centimeters with one ampoule of adhesive.
- Only one, thin layer of adhesive is needed for full strength closure.
- Flexibility of polymerized adhesive is critical to good cosmetic results.
- Beveled applicator tip provides enhanced control and improved application, for longer, wider incisions.

HISTOACRYL TISSUE SEAL COMPARISON

	Histoacryl Blue & Clear	Histoacryl Flexible
Composition	n-butyl 2 -cyanoacrylate	n-butyl 2 -cyanoacrylate w/ Plasticizers
Color	Blue & Clear	Blue
Indications	To hold closed clean surgical incisions and cleansed trauma induced lacerations	To hold closed clean surgical incisions and cleansed trauma included lacerations
Color Code <small>(Ampoule Packaging)</small>	White and Orange	White and Black
Storage	24 months @ 72°F	24 months @ 77°F
NSN#	6510-01-586-6762 (Histoacryl Clear) 6510-01-585-2101 (Histoacryl Blue)	N/A



APPLICATOR TIP
Only available with
Histoacryl Flexible

1051260-P
Histoacryl Flexible

TS1050071-FP
Histoacryl Clear

TS1050044-FP
Histoacryl Blue



MedLog Solutions, LLC

ORDERS/QUOTES:
T: 844-590-8787 | F: 281-408-4556
ORDER@MEDLOGSOLUTIONS.COM
MEDLOGSOLUTIONS.COM



A SERVICE-DISABLED VETERAN
OWNED SMALL BUSINESS
DUNS: 080705380 CAGE CODE: 7VWG2
FSS/GSA CONTRACT: 36F79720D0093
UEI: XEKUDLKN3HZ4
FEDERAL TAX ID: 82-1756285

IMPORTANT: The information contained herein is believed to be correct to the best of our knowledge. However the recommendations and suggestions herein are made without guarantee or representation as to results. It is the purchaser's responsibility to test and determine the suitability of the product for the purchaser's intended use and purpose. Purchaser assumes all risk and liability whatsoever regarding such suitability. Any product samples provided for testing are provided in accordance with standard limited warranties as stated on our technical data sheets.

HISTOACRYL® BLUE & CLEAR

- Ampoules - 10 / box

HISTOACRYL® FLEXIBLE

- Ampoules - 10 / box
- Applicator Tips - 10 / box

MICROBIAL BARRIER


In vitro studies have shown that Histoacryl Flexible acts as a barrier to microbial penetration as long as the adhesive film remains intact.*

- Flexibility of an Octyl, tensile strength of a Butyl
- More than 1200 publications site Histoacryl®
- One Histoacryl Flexible vile, closes 25 linear centimeters
- Histoacryl Flexible provides a microbial barrier for 7 days, with intact adhesive film
- Precision applicator with optional beveled tip, for longer, wider applications




Part #	Description	UOM	#/UOM
1051260-P	Histoacryl Flexible 10 / 0.5 mL ampoule 10 Applicator Tips	Box	10
TS1050044-FP	Histoacryl Classic - Blue 10 / 0.5 mL ampoule	Box	10
TS1050071-FP	Histoacryl Classic - Clear 10 / 0.5 mL ampoule	Box	10

OPEN

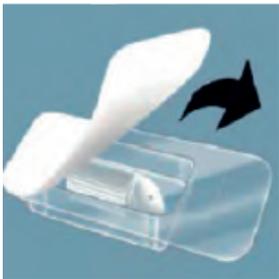

Open the package and take out the Histoacryl Tip.

PREPARE APPLICATOR TIP**

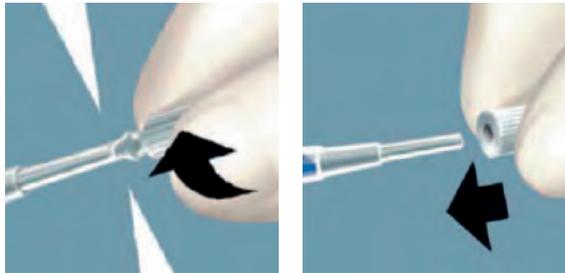

Open the ampoule by twisting off the ribbed tip. Attach the Histoacryl Tip to opened ampoule

APPLY



Apply the glue in a thin layer to the approximated wound edges and hold in position for 60 seconds.

The polymerization starts immediately upon contact with the tissue or fluid.



MedLog Solutions, LLC

ORDERS/QUOTES:
T: 844-590-8787 | F: 281-408-4556
ORDER@MEDLOGSOLUTIONS.COM
MEDLOGSOLUTIONS.COM



A SERVICE-DISABLED VETERAN OWNED SMALL BUSINESS
DUNS: 080705380 CAGE CODE: 7VWG2
FSS/GSA CONTRACT: 36F79720D0093
UEI: XEKUDLKN3HZ4
FEDERAL TAX ID: 82-1756285

*Histoacryl Flexible inhibits the growth of Staphylococcus aureus, Staphylococcus epidermidis, Escherichia coli, Pseudomonas aeruginosa, Enterococcus faecium, Brevundimonas dimuta and Candida albicans for up to 7 days on inoculated agar plates. **Applicator Tip only comes with Histoacryl Flexible.